Creating balance in healthcare

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6100 3070 Stylex Shower stretcher
Stainless steel

Important:

Read these instructions carefully before using the shower stretcher / dressing table!

Date first used

User

Name

Address

Manufacturer

Lopital Nederland B.V.
 Laarakkerweg 9
 5061 JR The Netherlands
 T (0031) 13 52 39 300
 F (0031) 13 52 39 301
 www.lopital.com
 info@lopital.nl

All of the aids produced by Lopital fulfil the following norms:
 MDD 93/42 EEG, NEN-EN-ISO 14971, NEN-EN 12183, ISO 17966, ISO 9001:2015, ISO 13485:2016/ C12:2018, NEN-EN 12182, ISO 60601, EN ISO 60601-1-2, EN ISO 60601-1-6, EN ISO 62366, CAN/CSA 60601, ANSI/AAMI ES 60601-1

Notes on symbols	
	Read the user instructions before use.
	Direct current 24V DC.
	Note.
	Applied parts.
	Class 2 EMC.

Contents

1.	Introduction.....	33
1.1.	Introduction.....	33
1.2.	Safety.....	34
1.3.	Warranty.....	36
2.	Use.....	37
2.1.	First use.....	37
2.2.	Activating (<i>Only for electrical adjustable versions</i>).....	37
2.3.	Working with the shower stretcher / dressing table.....	39
3.	Operation.....	40
3.1.	Wheel blocking.....	40
3.2.	Side bars.....	40
3.3.	Moving upwards / downwards.....	41
3.4.	Water drainage.....	42
3.5.	Charging battery (<i>Only for electrical adjustable versions</i>).....	42
4.	Safety functions.....	44
4.1.	Lock button (<i>Only for electrical adjustable versions</i>).....	44
4.2.	Sleep setting (<i>Only for electrical adjustable versions</i>).....	44
4.3.	Emergency lowering button (<i>Only for electrical adjustable versions</i>).....	44
4.4.	On / off switch (<i>Only for electrical adjustable versions</i>).....	45
5.	Cleaning and maintenance.....	45
6.	Storage and transport.....	46
7.	Removal of parts.....	47
8.	Technical specifications.....	47
8.1.	Structure.....	47
8.2.	Drive.....	49
8.3.	Dimensions.....	49
9.	Checklist for technical problems.....	52
10.	Relationship with other supports.....	53
11.	Electromagnetic compatibility.....	54

1. Introduction

1.1. Introduction

The shower stretcher / dressing table is a medical device for caring for and washing clients in an ergonomically responsible manner. Comfort is the primary consideration for both the client and the carer.

Lopital supplies various versions of its shower stretchers and dressing tables. The range includes stretchers / dressing tables that are secured to a wall and mobile shower stretchers / dressing tables. The height of some variants is electrically or hydraulically adjustable.

The shower stretcher / dressing table is multi-functional in use and corresponds to the use of other devices such as beds, wheelchairs and shower chairs, hoisting equipment and sliders such as the EasySlide. These are the essential functions of a shower stretcher / dressing table.

The average technical life of the shower stretcher / dressing table is 10 years, assuming it is cleaned daily and regularly maintained according to the maintenance schedule supplied (*See chapter 5 Cleaning and maintenance*).

The shower stretcher / dressing table is a class 1 product with CE-marking. This marking corresponds with directive 93/42/EC and the revised directive 2007/47/EC for medical devices of 21 March 2010. The shower stretcher / dressing table has been tested for electromagnetic radiation according to norm 60601-1-2.

The Amfora is available in the following versions:

- 6200 4010 – Hydraulic high / low, lying surface 160 cm / 63 inch.
- 6200 4015 – Hydraulic high / low, lying surface 190 cm / 74.8 inch.
- 6200 4011 – Electric high / low, lying surface 160 cm / 63 inch.
- 6200 4016 – Electric high / low, lying surface 190 cm / 74.8 inch.

The Vienna is available in the following versions:

- 6500 4020 – Hydraulic high / low, lying surface 160 cm / 63 inch.
- 6500 4025 – Hydraulic high / low, lying surface 190 cm / 74.8 inch.
- 6500 4021 – Electric high / low, lying surface 160 cm / 63 inch.
- 6500 4026 – Electric high / low, lying surface 190 cm / 74.8 inch.

The Stylex is available in the following versions:

- 6100 3070 – Standard stainless steel

1.2. Safety

A failure to comply with the following safety points and the further definitions in this user manual could lead to hazardous situations. Please read carefully before use.

Use

The following applies to all shower stretchers and dressing tables:

- It is not permitted to load the shower stretcher / dressing table with more weight than 135 kg / 297.6 lbs (Stylex) or 150 kg / 330.7 lbs (Amfora/Vienna).
- Only use original parts, supplied by Lopital.
- Use the shower stretcher / dressing table in a clean and tidy working environment.
- Only verifiably authorised personnel is allowed to work with the shower stretcher / dressing table.
- Swimming pool locations are not considered to be 'normal conditions' and will substantially shorten the lifespan of certain components.
- The shower stretcher / dressing table cannot be used on a surface with a gradient over 5 degrees laterally.
- The shower stretcher / dressing table cannot be used on a surface with a gradient of over 10 degrees forwards / backwards.
- The shower stretcher / dressing table must be used / cleaned using water that is below 43°C (109°F).
- The shower stretcher / dressing table may only be used indoors.

- Make sure that the shower stretcher / dressing table brake is applied before transferring clients.
- Notify the client before moving the shower stretcher / dressing table.
- Only relocate and move the shower stretcher / dressing table in the lower position.
- When moving the shower stretcher / dressing table, do not adjust upwards or downwards.
- Always try to move backwards over obstacles (rather than forwards).
- The shower stretcher / dressing table may not be used to transport loads other than people.
- Be aware of obstacles when moving upwards and downwards.

Electric adjustable:

- The use of components (ladders, cables, actuators, battery, etc.) that have not been supplied by Lopital may result in increased emissions or decreased immunity of the shower stretcher / dressing table.
- Charge battery in a well-ventilated, dry space.
- Portable and mobile RF communication equipment can affect medical devices.
- Medical electrical devices involve special EMC precautions and must be used as per the EMC information outlined in the instructions for use.
- The shower stretcher / dressing table must not be used adjacent to or stacked on other products. However, should that be necessitated, it must be checked whether the stretcher functions normally in this configuration too.

Maintenance

- If the charger or cable become damaged, they must be replaced. (For electric adjustable versions)
- The shower stretcher / dressing table must be inspected and serviced at least once a year.
- Servicing and maintenance of the shower stretcher / dressing table may only be carried out by persons authorised by Lopital.
- Constructive components of the shower stretcher / dressing table may only be replaced by suitably authorised Lopital personnel.
- If erroneous use, transport, an accident or improper maintenance leads to deformation of the shower stretcher / dressing table, it may no longer be used and you must contact the supplier.

In the event of breakdowns, you must contact Lopital or your local supplier.

E-mail: service@lopital.nl

The shower stretcher / dressing table must only be operated by persons who have been sufficiently trained and expert personnel who have experience working with the target group.

The shower stretcher / dressing table must be used in a clinical setting.

Making changes to the construction could affect the safety of the shower stretcher / dressing table. This will also lead to Lopital's liability and warranty conditions being invalidated. As a result, the shower stretcher / dressing table will no longer comply with the guidelines for medical devices.

1.3. Warranty

Lopital products are guaranteed for 2 years with respect to construction and / or material faults. In the event of varying warranty terms, the exact period will be indicated on your invoice; this also serves as the warranty itself.

Components that are subject to wear and tear (such as battery, hand controls and cables) will not fall under warranty unless there are clear construction and / or material faults.

2. Use

2.1. First use

Before you use the shower stretcher / dressing table, you must check to ensure the following components are present. These components can be replaced by authorised individuals in the event of damage. These components are the applied parts of the shower stretcher / dressing table. These components may come into contact with the user(s) while the shower stretcher / dressing table is in use.

2.2. Activating *(Only for electrically adjustable versions Amfora and Vienna)*

The shower stretcher / dressing table must be activated before use, which can be done by pressing any button on the hand control for 2 seconds.

2.3. Working with the shower stretcher / dressing table

Step 1

Always apply the brake on the shower stretcher / dressing table when transferring clients
(See paragraph 3.1. Wheel blocking).

Transfer to bed

Step 2

Place the shower stretcher / dressing table alongside the bed with the head-end aligned with the head-side of the client.

Step 3

Lower the side bar on the shower stretcher / dressing table (*See paragraph 3.2 Side-bars*).

Step 4

Move the bed base on the shower stretcher / dressing table to just above the bed mattress and collapse the side-bars downwards (*on adjustable shower stretchers / dressing tables*).

Step 5

Slide the shower stretcher / dressing table between the head and foot-end so that it protrudes as far as possible over the bed.

Step 6

Block the wheels on the shower stretcher / dressing table (*See paragraph 3.1. Wheel blocking*).

Step 7

Go to the other end of the bed and place the EasySlide between the client and the shower stretcher / dressing table.

Step 8

Use the EasySlide to slide the client from the bed to the shower stretcher / dressing table.

Step 9

Raise the horizontal surface of the shower stretcher / dressing table a few centimetres and close the side-bars.

Step 10

Take the brake off the shower stretcher / dressing table and remove the EasySlide (*See paragraph 3.1. Wheel blocking*).

3. Operation

3.1. Wheel blocking

- The brake can be applied on the shower stretcher / dressing table by pushing the blue lip (1) on the wheel downwards (See figure 1).
- In order to take the shower stretcher / dressing table off the brake, the same lip on the wheel must be moved upwards by tapping the upper side of the lip (2) (See figure 1).

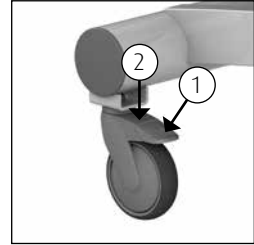
Figure 1 Wheel blocking.

3.2. Side bars

- The shower stretcher / dressing table is equipped with steel, hinged side bars. The side bars are made of stainless steel on the Stylex.
- The side-bar is equipped with safeguards so that the client cannot raise the side-bars himself.
- Before the side-bar is collapsed downwards, the safeguard must be unlocked.
- Unlock the safeguard by removing the bolts on both sides of the side bar (1)
- Turn the unlocked side bar outwards (3).
- Move the side-bar upwards in order to lock it in place again. The side bar is locked on the mobile shower stretcher / dressing table by securing the bolts in the appropriate position.

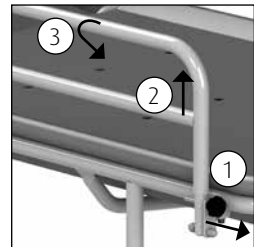
Figure 2 Side bar mobile stretchers / dressing tables.

3.3. Moving upwards / downwards

Electric adjustable versions

Moving upwards

- If the client is lying on the shower stretcher / dressing table, first put the brake on the wheels (if it is a mobile stretcher / dressing table) and then move the shower stretcher / dressing table upwards.
- Move the shower stretcher / dressing table upwards by pressing the button that shows an 'up' arrow. The shower stretcher / dressing table will then immediately move upwards (See figure 3).
- You can stop the movement whenever you like by taking your finger off the button.
- During the upward movement, maintain contact with the client.
- If the remote control is not being used, it can be hung on the side-bar.
- Ensure that the curly cable on the operating device does not get jammed.

Figure 3 Moving upwards.

Moving downwards

- Move the shower stretcher / dressing table downwards by pressing the button that shows a 'down' arrow. The shower stretcher / dressing table will then immediately move downwards (See figure 4).
- You can stop the movement whenever you like by taking your finger off the button.
- During the downward movement, maintain contact with the client.

Figure 4 Moving downwards.

Before moving the shower stretcher / dressing table upwards, check to ensure there are no persons or objects in the vicinity of the device.

Hydraulic adjustable versions

Moving upwards

- If the client is lying on the shower stretcher / dressing table, first put the brake on the wheels and then move the shower stretcher / dressing table upwards.
- The shower stretcher / dressing table moves gently and evenly upwards as a result of tapping the long part of the foot pedal (See figure 5).
- During use of the pedal, the carer will stand by the long edge of the shower stretcher / dressing table so that the carer cannot twist their upper body away from their feet.
- During the upward movement, maintain contact with the client.

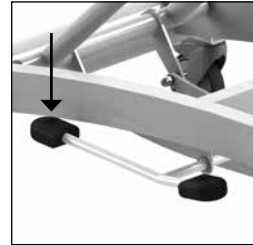
Figure 5 Moving upwards.

Moving downwards

- The shower stretcher / dressing table moves downwards as a result of pushing the short part of the foot pedal downwards (See figure 6).
- The shower stretcher / dressing table lowers under the influence of gravity. The lowering speed can be modified by keeping the pedal gently depressed.
- During the downward movement, maintain contact with the client.

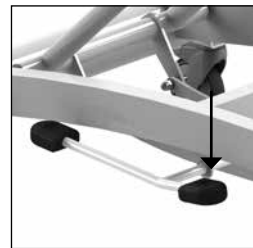
Figure 6 Moving downwards.

3.4. Water drainage

- Water is drained away from the shower stretcher via a drainage hose.
- Take the hose off the hook and direct it to a drainage point that is as low as possible.
- During transport, always hang the hose on the hook otherwise it may leak and drip.

3.5. Charging battery

(Only for electrically adjustable versions)

- The shower stretcher / dressing table is an electrically driven high / low stretcher and works on a battery.
- The charging point is located in the centre of the undercarriage (See figure 7).
- If the battery is almost completely empty of charge, the orange LED on the manual control will come on; this means that the battery needs to be charged.
- If the LED is on continuous red, the battery must be charged immediately. The shower stretcher / dressing table can then no longer be used.

- Connect the charger to the shower stretcher / dressing table (See figure 7).
- Once the charger is connected to the shower stretcher / dressing table, the controls will go into 'sleep' setting (all functions are switched off).

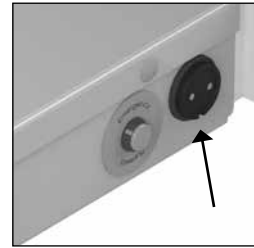
Figure 7 Charging point.

Make sure that you push the plug into the correct position in the charging point.

Only the original charger for the shower stretcher / dressing table may be used. If you use another charger, Lopital cannot be held liable for any consequences, such as damage to the electronics.

Once the charger is connected to the shower stretcher / dressing table, the controls will go into 'sleep' setting (all functions are switched off).

The charger that must be used for the shower stretcher / dressing table is the 'Mascot 2241'.

- When the indicator light on the charger goes green, the battery is charged. The charger can remain connected even when the battery is completely charged. Leaving it connected will not cause any damage.
- Disconnect the shower stretcher / dressing table from the charger.
- Once activated, the shower stretcher / dressing table is now ready for use (See paragraph 2.2 *Activating*).

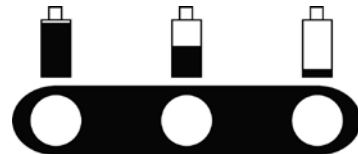
Figure 8 Battery indication on handcontrol.

- LEDs not on:** Shower stretcher / dressing table is in 'sleep' setting and cannot be used or the on / off switch is off.
- Green LED is on:** Shower stretcher / dressing table is ready for use.
- Orange LED blinks:** Lock button has been activated.
- Orange LED is on:** Battery needs charging or battery is partially charged.
- Red LED blinks:** Battery must be charged (shower stretcher / dressing table can still be used).
- Red LED on:** Battery is completely empty and must be charged immediately.

Charge battery in a well-ventilated, dry space.

After using the shower stretcher / dressing table, connect it to the charger to maintain the battery. Charge the stretcher at least once a week!

For instruction videos go to www.lopital.com or www.youtube.com/user/LopitalNL

4. Safety functions

4.1. Lock button *(Only for electrically adjustable versions)*

The lock button is located in the middle of the undercarriage. Once the lock button has been depressed all electrical functioning will stop. Rotate the lock button a quarter of a turn to the right to unlock it (See figure 9).

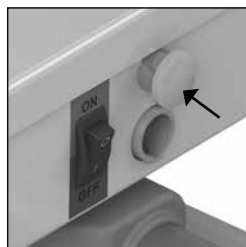
Figure 9 Lock button.

The orange LED light on the hand control will flash when the Lock button has been activated.

4.2. Sleep setting *(Only for electrically adjustable versions)*

After around an hour, the stretcher will fall into a 'sleep' setting and the battery indication LEDs on the manual control will go out. In order to use the stretcher again, it must be activated (See paragraph 2.2 Activating).

4.3. Emergency lowering

(Only for electrically adjustable versions)

If the controls stop working and the stretcher is in a raised position, it can be lowered using the emergency lowering button (See figure 10).

Check first to make sure the stretcher is not in sleep setting!

- Remove the plug from the manual control (1).
- Press the emergency lowering button (2).

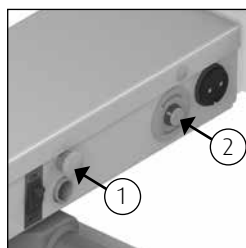
Figure 10 Emergency lowering button.

The stretcher will now return to the lowest position (make sure no persons or objects are under the stretcher). Take the client from the stretcher and make sure the device is not used until it has been repaired. Place the stretcher in a safe place and indicate that it is out of order.

Contact the technical service or Lopital's service department.
E-mail: service@lopital.nl

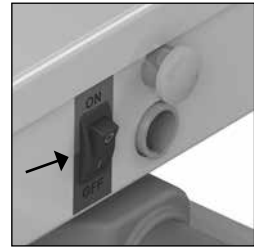
Figure 11 On / off switch.

4.4. On / Off switch

(Only for electrically adjustable versions)

In the centre of the undercarriage, there is an on / off switch. If the shower stretcher is not used for an extended period, you are advised to switch it off.

5. Cleaning and maintenance

The shower stretcher / dressing table must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the lying surface can be disinfected with a solution of 70% ethanol or propanol.

Check the wheels regularly for hairs and dust and remove when necessary. Maintenance and repairs to the shower stretcher / dressing table may only be carried out by authorised personnel.

Maintenance schedule	
After every use	<ul style="list-style-type: none"> • The shower stretcher / dressing table can be cleaned with water and regular household cleaning products. • Clean and disinfect mattress / harness.
Weekly	<ul style="list-style-type: none"> • Clean frame. • Check wheels for hairs, dust and soap residues and remove if necessary. • Check that the shower stretcher / dressing table is still working correctly.
<p>Annually</p> 	<ul style="list-style-type: none"> • According to the guidelines for medical devices, a shower stretcher / dressing table must be maintained in line with the manufacturer's instructions. This process will preferably be carried out by Lopital or another authorised body. Lopital offers the option of having the shower stretcher / dressing table thoroughly inspected and serviced once a year via Periodic Preventative Maintenance. <p>For more information about the maintenance contract, please contact Lopital (service@lopital.nl) or speak to your supplier.</p>

6. Storage and transport

During transport and storage, the ambient conditions must remain moderate. In other words, the temperature should remain between -40°C and +55°C (-40°F to 130°F) and humidity levels should be kept as low as possible. Vibration during transport must be kept to a minimum in order to avoid damaging the hinged mechanisms on the stretcher.

7. Removal of parts

All parts that are replaced or removed can be sent back to Lopital BV. You may also offer the parts to the engineer for destruction. We will ensure that all parts are processed in an environmentally-friendly manner.

If you take care of disposal of the product yourself, you must take account of the fact that battery and chargers must be disposed of as small chemical waste.

8. Technical specifications

This shower stretcher / dressing table belongs to category 'BF'.

This shower stretcher / dressing table is a class 2 EMC product

The shower stretcher / dressing table is splash-proof; this means that it can be used under the shower. This is indicated with instruction IPX4.

8.1. Structure

Amfora & Vienna mobile:

Subframe	Aluminium + 2-layer powder coating
Upper frame	Steel + 2-layer powder coating
Side bars	Steel + 2-layer powder coating
Hinges	Stainless steel + 2-layer powder coating

Stylex:

Subframe	Stainless steel + 2-layer powder coating
Upper frame	Stainless steel + 2-layer powder coating
Side bars	Stainless steel + 2-layer powder coating
Hinges	Stainless steel + 2-layer powder coating

Mobile stretcher / dressing table:

Wheels Ø 12.5 cm / 4.9 inch, double blocking

Attachment equipment Stainless steel + galvanised steel

Weight

Amfora

6200 4010/15 60 / 63 kg 132.3 / 138.9 lbs

6200 4011/16 61 / 64 kg 134.5 / 141.1 lbs

Vienna

6500 4020/25 56 / 59 kg 123.5 / 130.1 lbs

6500 4021/26 60 / 63 kg 132.3 / 138.9 lbs

Stylex

6100 3070 50 kg 110.2 lbs

Maximum load

135 kg / 297.6 lbs (*Stylex*)

150 kg / 330.7 lbs (*Amfora / Vienna*)

8.2. Drive

(Electric variants)

High-low drive	Actuator 24 V DC
Control box	24 VDC, max. 250W
Battery	24 V, max. 250 W
Battery charger	230 V AC, connection value 100 W, wall-mounted

8.3. Dimensions

External dimensions

Amfora (l x w x h)

6200 4010/15	160 / 190 x 74 x 50 to 100 cm	63 / 74.8 x 29.1 x 19.7 to 39.4 inch
6200 4011/16	160 / 190 x 74 x 50 to 100 cm	63 / 74.8 x 29.1 x 19.7 to 39.4 inch





Vienna (l x w x h)

6500 4020/25	160 / 190 x 74 x 56 to 106 cm	63 / 74.8 x 29.1 x 22.0 to 41.7 inch
6500 4021/26	160 / 190 x 74 x 56 to 106 cm	63 / 74.8 x 29.1 x 22.0 to 41.7 inch

Stylex (l x w x h)

6100 3070 – 194 x 72,5 x 52 tot 95 cm	764 x 28.3 x 20.5 to 37.4 inch
---------------------------------------	--------------------------------

Water drainage hose	Internal Ø 4 cm / 1.6 inch
Wheel base	120 cm / 47.2 inch
Track width	65 cm / 25.6 inch

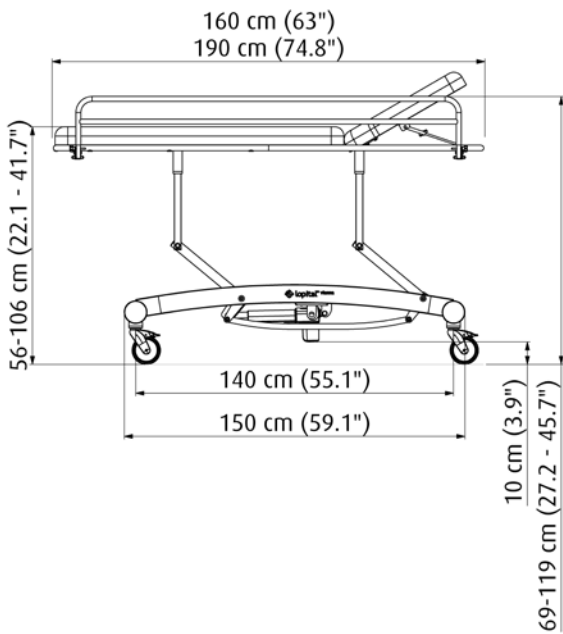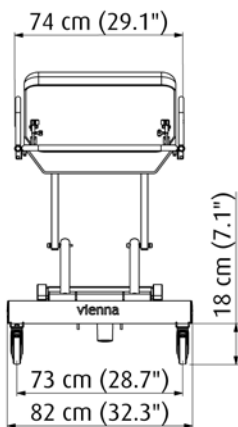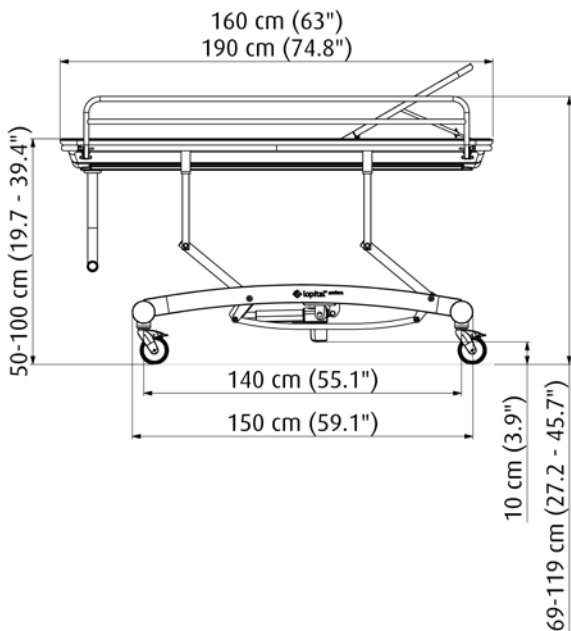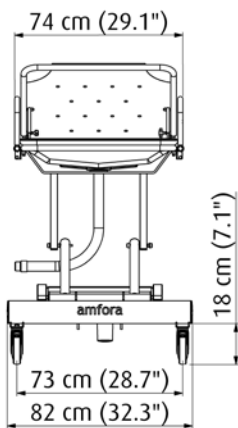9. Checklist for technical problems

Problem	Cause	Solution
Shower stretcher / dressing table is hard to move	Dirt and soap between wheels.	Clean wheels.
Shower stretcher / dressing table does not move upwards or downwards	Emergency stop button used.	Turn emergency button 90 degrees to the right.
	Battery empty.	Charge battery / install reserve battery.
	Plugs removed.	Check plugs (to side of control box).
Light on the charger does not come on	Manual controls not working.	Check plugs. Contact Lopital Nederland B.V.
	Charger is not plugged in.	Connect charger.
Shower stretcher dressing table stops	Charger is not working.	Contact Lopital Nederland B.V.
	Lock button used.	Turn lock button 90 degrees to the right.
	Technical issue.	Contact Lopital Nederland B.V.

For all other issues please contact Lopital Nederland B.V.

Telephone: +31 (0)13 - 5239300

Fax: +31 (0)13 - 5239301

E-mail: service@lopital.nl

Guidance and Manufacturer's declaration-electromagnetic emissions		
The shower stretcher / dressing table is intended for use in the electromagnetic environment specified below. The customer or the user of the shower stretcher / dressing table should assure that it is used in the following environment:		
Emissions Test	Compliance	Electromagnetic Environment - guidance
RF emissions CISPR11	Group 1	The shower stretcher / dressing table uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR11	Class B	The shower stretcher / dressing table is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/ Flicker emissions IEC 61000-3-3	Complies	

Guidance and Manufacturer's declaration-electromagnetic immunity			
The shower stretcher / dressing table is intended for use in the electromagnetic environment specified below. The customer or the user of the shower stretcher / dressing table should assure that it is used in the following environment:			
Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be made of wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) to line(s) ±2 kV line to earth	±1 kV line(s) to line(s) ±2 kV line to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions, and voltage variations on power supply input lines IEC 61000-4-11	<5% Un (>95% drop in Un) for 0,5 cycle 40% Un (60% drop in Un) for 5 cycles 70% Un (30% drop in Un) for 25 cycles <5% Un (>95% drop in Un) for 5 seconds	<5% Un (>95% drop in Un) for 0.5 cycle 40% Un (60% drop in Un) for 5 cycles 70% Un (30% drop in Un) for 25 cycles <5% Un (>95% drop in Un) for 5 seconds	Mains power quality should be that of a typical commercial or hospital environment. The shower stretcher / dressing table is equipped with a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE Un is the a.c. mains voltage prior to application of the test level			

Guidance and Manufacturer's declaration-electromagnetic immunity			
The shower stretcher / dressing table is intended for use in the electromagnetic environment specified below. The customer or the user of the shower stretcher / dressing table should assure that it is used in the following environment:			
Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment - guidance
RF Common mode/ Conducted Susceptibility IEC 61000-4-6 Radiated RF Electromagnetic Field IEC 61000-4-3	3 Vrms 150 kHz to 80 MHz 3 V/m 80 MHz to 2,5 GHz	3 V 3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the shower stretcher / dressing table, including cables than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2,4 \sqrt{P}$ 800 MHz to 2,5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the shower stretcher / dressing table is used exceeds the applicable RF compliance level above, the shower stretcher / dressing table should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the shower stretcher / dressing table. b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Recommended separation distance between portable and mobile RF communications equipment and the shower stretcher / dressing table			
The shower stretcher / dressing table is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the shower stretcher / dressing table can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the shower stretcher / dressing table as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2 \sqrt{P}$	80 MHz to 800 MHz $d = 1,2 \sqrt{P}$	800 MHz to 2,5 GHz $d = 2,4 \sqrt{P}$
0,01	0,12	0,12	0,24
0,1	0,38	0,38	0,76
1	1,2	1,2	2,4
10	3,8	3,8	7,6
100	12	12	24
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watt (W) according to the transmitter manufacturer.			
NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

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Guidance and Manufacturer's declaration-electromagnetic immunity			
The shower stretcher / dressing table is intended for use in the electromagnetic environment specified below. The customer or the user of the shower stretcher / dressing table should assure that it is used in the following environment:			
Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment - guidance
<p>RF Common mode/ Conducted Susceptibility IEC 61000-4-6</p> <p>Radiated RF Electromagnetic Field IEC 61000-4-3</p>	<p>3 Vrms 150 kHz to 80 MHz</p> <p>3 V/m 80 MHz to 2,5 GHz</p>	<p>3 V</p> <p>3 V/m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the shower stretcher / dressing table, including cables than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance</p> <p>$d = 1,2 \sqrt{P}$</p> <p>$d = 1,2 \sqrt{P}$ 80 MHz to 800 MHz</p> <p>$d = 2,4 \sqrt{P}$ 800 MHz to 2,5 GHz</p> <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			
<p>a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the shower stretcher / dressing table is used exceeds the applicable RF compliance level above, the shower stretcher / dressing table should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the shower stretcher / dressing table.</p> <p>b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.</p>			

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Guidance and Manufacturer's declaration-electromagnetic immunity			
The shower stretcher / dressing table is intended for use in the electromagnetic environment specified below. The customer or the user of the shower stretcher / dressing table should assure that it is used in the following environment:			
Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment - guidance
<p>RF Common mode/ Conducted Susceptibility IEC 61000-4-6</p> <p>Radiated RF Electromagnetic Field IEC 61000-4-3</p>	<p>3 Vrms 150 kHz to 80 MHz</p> <p>3 V/m 80 MHz to 2,5 GHz</p>	<p>3 V</p> <p>3 V/m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the shower stretcher / dressing table, including cables than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> <p>$d = 1,2 \sqrt{P}$</p> <p>$d = 1,2 \sqrt{P}$ 80 MHz to 800 MHz</p> <p>$d = 2,4 \sqrt{P}$ 800 MHz to 2,5 GHz</p> <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			
<p>a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the shower stretcher / dressing table is used exceeds the applicable RF compliance level above, the shower stretcher / dressing table should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the shower stretcher / dressing table.</p> <p>b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.</p>			

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For more information, visit our website: www.lopital.com

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Für alle weiteren Störungen nehmen Sie bitte Kontakt mit Lopital Nederland B.V. auf.
Lopital Nederland B.V. Laarakkerweg 9, 5061 JR Oisterwijk, Postbus 56, 5060 AB Oisterwijk