

Z:\SW-Tek\Peilkaan lijn\4070\

| | | | |
|---------|-------|------|------------|
| Signed | PS | Date | 17-11-2005 |
| Revised | 2 /PS | Date | 19-12-2008 |
| Revised | 3 /KA | Date | 22-01-2015 |

Copyright reserved by law Unit of measurement: mm


Overview parts Vienna

Lopital Nederland bv

Type nr: 65004070/4075

65004070 / 65004075

| Part nr. | QTY. | Art.nr. | Part |
|----------|------|----------------------|--|
| 1 | 2 | 70403004 | Leg |
| 2 | 1 | 70407003 70407502 | Frame L=160 Frame L=190 |
| 3 | 1 | 67904120 | Side rail complete L=160 |
| 4 | 2 | 70407002 | Wallmounting L=95 |
| 5 | 1 | 70407001 | Locking support long |
| 6 | 2 | 70411003 | Hinge for back |
| 7 | 1 | 70407005 70407505 | Matress set L=160 Matress set L=190 |
| 8 | 6 | 70402003 | Stainless steel clip |
| 9 | 1 | 70403005 | Locking axle |
| 10 | 2 | 70411004 | Hook support |
| 11 | 2 | 70411002 | Tube back adjusting |
| 12 | 2 | | Buffer cap |
| 13 | 10 | | Nut cap |
| 14 | 2 | | Hex bolt |
| 15 | 24 | | Washer |
| 16 | 6 | | Lock nut |
| 17 | 6 | | Chipboard screw |
| 18 | 8 | | Sluitring |
| 19 | 4 | | Nut cap |
| 20 | 2 | | Bolt |
| 21 | 2 | | Locknut |
| 22 | 2 | | Ball knob |
| 23 | 8 | | Hex bolt |
| 24 | 2 | | Hex bolt |
| 25 | 2 | | Wingbolt |

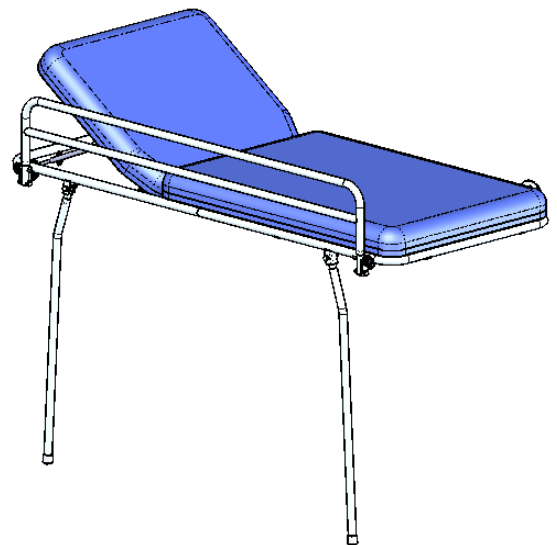

Z:\SW-Tek\Peilkaan lijn\4070\

| | | | |
|---------|--------------|------|------------|
| Signed | PS | Date | 17-11-2005 |
| Revised | 2 /PS | Date | 19-12-2008 |
| Revised | 3 /KA | Date | 22-01-2015 |

Overview parts Vienna

Copyright reserved
by law

Unit of measure-
ment: mm

Lopital Nederland bv

Type nr: 65004070/4075